

NITRADINE®

Patented disinfecting formula

Patented disinfecting tablet for dentures & removable dental appliances.

Effective against *Candida Albicans*, *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Herpes Simplex Virus*.



PATENTED FORMULA

Ingredients: Citric Acid, Sodium Lauryl Sulfate, Lactose Monohydrate, Sodium Bicarbonate, Sodium Chloride, Potassium Monopersulfate, Sodium Carbonate, Peppermint Flavour, PVP.

Manufactured by:

bonyf AG   0124

Heiligkreuz 16
FL-9490 Vaduz
Liechtenstein - EUROPE
www.bonyf.com

Distributed by:

W H W Plastics Ltd

Therm Rd, Cleveland St
HULL HU8 7BF
UK - EUROPE
www.whwplastics.com

whw
Behind every smile

Instructions for use (dentures & removable dental appliances)



Before using the NitrAdine® tab, brush, cleanse and rinse the appliance or denture as usual.



150 ml

Immerse the appliance or denture in a glass of lukewarm water (approx. 150ml).



1 Tablet

Add one tablet. Use only one upper or one lower appliance or denture with one tablet.



15 min.

Make sure the appliance or denture is fully covered with water. For regular bacteria and yeast infection, allow the appliance or denture to soak for 15 minutes in the solution. To ensure that your appliance or denture is free of all viruses, soak it for 1 hour in the solution.



Remove the appliance or denture from the solution and rinse well under water before replacing it in the mouth. If a taste should remain on your appliance or denture after replacing it in your mouth, place it in lukewarm water for 10 minutes.

Recommended dosage: One tablet two times per week, in conjunction with the daily use of NitrAdine® SHINE Removable Appliance Toothpaste.

Intensive treatment: If recommended by your dentist or Orthodontist, use the tablet initially every day for two weeks. After this two week period has elapsed, use two times per week as part of your cleansing routine.

CAUTION

For external use only. Discard solution after use. Do not use if allergic to any of the ingredients or to persulfates. Do not put tablet in mouth. Tablet powder may cause eye damage. Do not drink the solution or use as a mouthwash. Rinse denture well before replacing it in your mouth. Discontinue use and seek medical attention if any irritation, swelling or pain develops following the first or any later use. Consult instructions for use. Store in a dry place. For single use only.



Dry tablet: Causes skin irritation. Causes serious eye damage. Wash hands thoroughly after handling. Keep out of reach of children. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention. Contains Pentapotassium bis(peroxymonosulphate) bis(sulphate). Contains Dipotassium peroxodisulphate. May produce an allergic reaction. Contains Commint oil/Mentha arvensis oil. May produce an allergic reaction.

Triple action of the NitrAdine® formula



Penetrates the microporosities of the appliance

Eliminates the micro-organisms

Removes the micro-organisms

ISO 13727 Bactericidal Activity in 15 minutes: Significantly reduces *Pseudomonas aeruginosa*, *Staphylococcus aureus* and *Enterococcus hirae*

ISO 13624 Yeasticidal activity in 15 minutes: Reduces *Candida albicans*

ISO 14476 + A1 Virucidal activity: Significantly reduces Adenovirus and Poliovirus in 1 hour

A single use of NitrAdine® is effective in eliminating certain species of microorganisms, including selected viruses (*Herpes Simplex*), *in vitro*. (Quintessence Int 2004, Prof. R. Thomas Glass, et al.)

A novel disinfecting formula for the maintenance of oral medical devices. (Journal of Applied Microbiology, Prof. T. Coenye)

